




## Easyboot & EasyShoe Size Charts

### How to Measure

The biggest determining factor in EasyCare hoof boot\* and EasyShoe success is fit. Always measure as soon as possible after a fresh trim. Waiting to take measurements can result in a poor fit.

Check the product size chart before starting your measurements. If the sizes are shown in millimeters only, be sure to measure in millimeters. Measuring in inches and converting to millimeters will render the measurement process inaccurate.

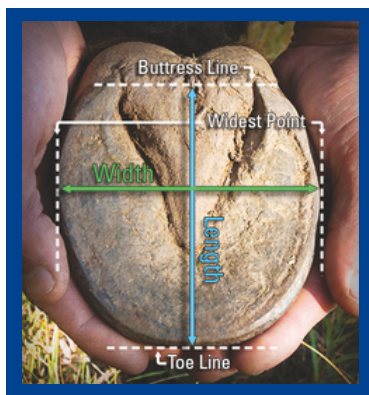

1. Measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurements with the appropriate size chart. Please note: each model has its own unique size chart.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are more than one size apart, your selected model is not recommended. \*\* Check your measurements against an alternative product in the EasyCare line - another model may accommodate your horse's hoof shape better.

\* Hoof boots should be snug before tightening. After tightening, the boot should be a close fit around the hoof wall. You should not be able to turn or twist the boot.

\*\* If the hoof measurements are more than one size apart in length and width, please call an EasyCare Product Specialist at 800.447.8836.

### Chart Reference

PERFORMANCE	
Easyboot Epic	1
Easyboot Flip Flop	2
Easyboot Fury	3
Easyboot Glove Soft	4
Easyboot Glove 50	5
Easyboot Glue-On	4
Easyboot Love Child (LC)	6
PLEASURE	
Easyboot Back Country	4
Easyboot Mini	7
Old Mac's G2	8
Easyboot Sneaker	9
Easyboot Trail	8
Easyboot Trail Original	8
THERAPY	
Easyboot Cloud	10
Easyboot Remedy	11
Easyboot Remedy Ultimate	11
Easyboot Rx	12
Easyboot Zip	13
EASYSHOE	
EasyShoe Compete	14
EasyShoe Flex Heart Bar	15
EasyShoe Performance	16
EasyShoe Performance N/G	16
EasyShoe Sport	17
EasyShoe Versa Grip	18
EasyShoe Versa Grip With Studs	18
EasyShoe Versa Grip Gold	18
EasyShoe Versa Grip Light	18
EasyShoe Versa Grip Glue	19
EasyShoe Versa Grip Octo	19
EasyShoe Versa Grip Wedge	20
EasyShoe Versa Grip Rocker	21
EasyShoe 3D	22

### CHART 1: Easyboot Epic

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>Pony</b>	3 3/16 - 3 3/8	3 3/8 - 3 5/8	78-85	85-92
<b>000</b>	3 3/8 - 3 13/16	3 5/8 - 4 3/16	86-97	93-105
<b>00</b>	3 13/16 - 4 3/8	4 3/16 - 4 5/8	98-110	106-117
<b>0</b>	4 3/8 - 4 9/16	4 5/8 - 4 15/16	111-116	118-124
<b>1</b>	4 9/16 - 4 7/8	4 15/16 - 5 1/8	117-123	125-130
<b>2</b>	4 7/8 - 5 1/8	5 1/8 - 5 9/16	124-130	131-140
<b>3</b>	5 1/8 - 5 1/2	5 9/16 - 5 15/16	131-139	141-150
<b>4</b>	5 1/2 - 5 15/16	5 15/16 - 6 7/16	140-150	151-163
<b>5</b>	5 15/16 - 6 7/16	6 7/16 - 6 7/8	151-163	164-174
<b>6</b>	6 7/16 - 6 7/8	6 7/8 - 7 7/16	164-175	175-188
<b>7</b>	6 7/8 - 7 9/16	7 7/16 - 7 3/4	176-192	189-195

### CHART 2: Easyboot Flip Flop

Size	Width (mm)	Length (mm)
<b>0</b>	104-107	Trim to size
<b>0.5</b>	108-111	Trim to size
<b>1</b>	112-115	Trim to size
<b>1.5</b>	116-119	Trim to size
<b>2</b>	120-123	Trim to size
<b>2.5</b>	124-127	Trim to size
<b>3</b>	128-131	Trim to size

### CHART 3: Easyboot Fury

Size	Width (mm)	Length (mm)
<b>0</b>	104-109	102-120
<b>0.5</b>	108-113	108-125
<b>1</b>	112-117	112-130
<b>1.5</b>	116-121	116-135
<b>2</b>	120-125	120-140
<b>2.5</b>	124-129	124-145
<b>3</b>	128-133	128-150
<b>3.5</b>	132-137	132-155
<b>4</b>	136-140	136-160

### CHART 4: Easyboot Glove Soft, Back Country, Glue-On

Regular Sizes		
Size	Width (mm)	Length (mm)
<b>00</b>	96-99	106-109
<b>00.5</b>	100-103	110-113
<b>0</b>	104-107	114-117
<b>0.5</b>	108-111	118-121
<b>1</b>	112-115	122-125
<b>1.5</b>	116-119	126-129
<b>2</b>	120-123	130-133
<b>2.5</b>	124-127	134-137
<b>3</b>	128-131	138-141
<b>3.5</b>	132-135	142-145
<b>4</b>	136-139	146-149
<b>4.5</b>	140-143	150-153
<b>5</b>	144-147	154-157

Wide Sizes		
Size	Width (mm)	Length (mm)
<b>0W</b>	114-117	114-117
<b>0.5W</b>	118-121	118-121
<b>1W</b>	122-125	122-125
<b>1.5W</b>	126-129	126-129
<b>2W</b>	130-133	130-133
<b>2.5W</b>	134-137	134-137
<b>3W</b>	138-141	138-141

Always measure in millimeters as directed in our "How to Measure" guide. A Fit Kit is available to ensure correct sizing.

### CHART 5: Easyboot Glove 50

Size	Width (mm)	Length (mm)
<b>0.5</b>	108-111	118-121
<b>1</b>	112-115	122-125
<b>1.5</b>	116-119	126-129
<b>2</b>	120-123	130-133
<b>2.5</b>	124-127	134-137

### CHART 6: Easyboot Love Child (LC)

Size	Width (mm)	Length (mm)
<b>0</b>	104-107	111-118
<b>0.5</b>	108-111	116-123
<b>1</b>	112-115	121-128
<b>1.5</b>	116-119	126-133
<b>2</b>	120-123	132-139
<b>2.5</b>	124-127	134-141
<b>3</b>	128-131	139-146
<b>3.5</b>	132-135	142-149
<b>4</b>	136-139	145-152

### CHART 7: Easyboot Mini

Size	Width (mm)	Length (mm)
<b>Mini</b>	44-49	49-54
<b>Mini 1</b>	50-55	55-64
<b>Mini 2</b>	56-65	65-72
<b>Mini 3</b>	66-76	73-86

### CHART 8: Old Mac's G2, Easyboot Trail & Trail Original

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>0</b>	3 15/16 - 4 1/8	4 1/8 - 4 15/16	100-105	105-110
<b>1</b>	4 1/8 - 4 5/16	4 5/16 - 4 1/2	105-110	110-115
<b>2</b>	4 5/16 - 4 1/2	4 1/2 - 4 3/4	110-115	115-120
<b>3</b>	4 1/2 - 4 3/4	4 3/4 - 4 15/16	115-120	120-125
<b>4</b>	4 3/4 - 4 15/16	4 15/16 - 5 1/8	120-125	125-130
<b>5</b>	4 15/16 - 5 1/8	5 1/8 - 5 5/16	125-130	130-135
<b>6</b>	5 1/8 - 5 5/16	5 5/16 - 5 1/2	130-135	135-140
<b>7</b>	5 5/16 - 5 1/2	5 1/2 - 5 11/16	135-140	140-145
<b>8</b>	5 1/2 - 5 11/16	5 11/16 - 5 15/16	140-145	145-150
<b>9</b>	5 11/16 - 5 15/16	5 15/16 - 6 1/8	145-150	150-155
<b>10</b>	5 15/16 - 6 1/8	6 1/8 - 6 5/16	150-155	155-160
<b>11</b>	6 1/8 - 6 1/2	6 5/16 - 6 11/16	155-165	160-170
<b>12</b>	6 1/2 - 6 7/8	6 11/16 - 7 1/8	165-175	170-180

### CHART 9: Easyboot Sneaker

Regular Sizes		
Size	Width (mm)	Length (mm)
<b>00</b>	90-100	95-105
<b>0</b>	100-110	105-115
<b>1</b>	110-120	115-125
<b>2</b>	120-130	125-135
<b>3</b>	130-140	135-145
<b>4</b>	140-150	145-155
<b>5</b>	150-160	155-165

Narrow Sizes		
Size	Width (mm)	Length (mm)
<b>00</b>	80-90	95-105
<b>0</b>	90-100	105-115
<b>1</b>	100-110	115-125
<b>2</b>	110-120	125-135
<b>3</b>	120-130	135-145
<b>4</b>	130-140	145-155
<b>5</b>	140-150	155-165

### CHART 10: Easyboot Cloud

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>00</b>	3 1/2 - 3 7/8	4 - 4 3/8	89-99	102-112
<b>0</b>	3 7/8 - 4 1/4	4 3/8 - 4 5/8	99-108	112-118
<b>1</b>	4 1/4 - 4 7/8	4 5/8 - 5 1/8	108-124	118-131
<b>2</b>	4 7/8 - 5 1/4	5 1/8 - 5 3/8	124-133	131-137
<b>3</b>	5 1/4 - 5 7/8	5 3/8 - 5 7/8	133-149	137-150
<b>4</b>	5 7/8 - 6 1/4	5 7/8 - 6 1/4	149-159	150-159
<b>5</b>	6 1/4 - 6 11/16	6 1/4 - 6 11/16	159-170	159-170
<b>6</b>	6 11/16 - 6 15/16	6 11/16 - 6 15/16	170-177	170-177
<b>7</b>	6 15/16 - 7 3/8	6 15/16 - 7 3/8	177-188	177-188
<b>8</b>	7 3/8 - 7 1/2	7 3/8 - 7 7/8	188-191	188-201

### CHART 11: Easyboot Remedy & Remedy Ultimate

Size	Width (in)	Width (mm)
<b>Medium</b>	Up to 4 3/4"	Up to 120.5
<b>Large</b>	Up to 6 1/2"	Up to 160

### CHART 12: Easyboot Rx

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>00</b>	3 13/16 - 4 3/8	4 3/16 - 4 5/8	98-110	106-117
<b>0</b>	4 3/8 - 4 9/16	4 5/8 - 4 15/16	111-116	118-124
<b>1</b>	4 9/16 - 4 7/8	4 15/16 - 5 1/8	117-123	125-130
<b>2</b>	4 7/8 - 5 1/8	5 1/8 - 5 9/16	124-130	131-140
<b>3</b>	5 1/8 - 5 1/2	5 9/16 - 5 15/16	131-139	141-150
<b>4</b>	5 1/2 - 5 15/16	5 15/16 - 6 7/16	140-150	151-163
<b>5</b>	5 15/16 - 6 7/16	6 7/16 - 6 7/8	151-163	164-174
<b>6</b>	6 7/16 - 6 7/8	6 7/8 - 7 7/16	164-175	175-188
<b>7</b>	6 7/8 - 7 9/16	7 7/16 - 7 3/4	176-192	189-195

### CHART 13: Easyboot Zip

Size	Width (in)	Length (in)	Width (mm)	Length (mm)
<b>0</b>	3 - 3 13/16	3 - 3 13/16	77-97	77-97
<b>1</b>	3 7/8 - 4 11/16	3 7/8 - 4 11/16	98-119	98-119
<b>2</b>	4 3/4 - 5 5/16	4 3/4 - 5 5/16	120-136	120-136
<b>3</b>	5 3/8 - 6 3/8	5 3/8 - 6 3/8	137-162	137-162
<b>4</b>	6 7/16 - 7 1/16	6 7/16 - 7 1/16	163-180	163-180

### CHART 14: EasyShoe Compete

Size	Width (mm)	Length (mm)
<b>0</b>	95-118	111-118
<b>1</b>	105-125	119-127
<b>2</b>	115-135	128-136
<b>3</b>	125-145	137-143

### CHART 15: EasyShoe Flex Heart Bar

Size	Width (mm)	Length (mm)
<b>0</b>	100-113	127
<b>1</b>	106-121	135
<b>2</b>	110-129	144
<b>3</b>	116-137	153
<b>4</b>	122-143	160
<b>5</b>	124-148	168
<b>6</b>	134-158	175

### CHART 16: EasyShoe Performance & Performance N/G

Size	Width (mm)	Length (mm)
<b>0</b>	105-111	108-115
<b>1</b>	112-119	116-123
<b>2</b>	120-127	124-131
<b>3</b>	128-135	132-140
<b>4</b>	136-143	141-149
<b>5</b>	144-152	150-159

**CHART 17: EasyShoe Sport**

Size	Width (mm)	Length (mm)
0	100-110	114-117
0.5	105-115	118-121
1	110-120	122-125
1.5	115-125	126-129
2	120-130	130-133
2.5	125-135	134-137
3	130-140	138-141

**CHART 18: EasyShoe Versa Grip, Versa Grip W/ Studs, Versa Grip Gold, & Versa Grip Light**

Front Sizes			
Size	Width (mm)	Length (mm)	Thickness (mm)
102F	96-102	113	16
106F	100-106	117	16
110F	104-110	122	16
114F	108-114	126	16.2
118F	112-118	131	16.2
122F	116-122	135	16.2
126F	120-126	139	16.2
130F	124-130	143	16.2
134F	128-134	148	16.2
138F	132-138	153	16.7
142F	136-142	157	16.7
146F	140-146	161	16.7
150F	144-150	166	17.2
154F	148-154	170	17.2
158F	152-158	175	17.2
162F	156-162	179	17.8
166F	160-166	183	17.8
170F	164-170	188	17.8

Hind Sizes			
Size	Width (mm)	Length (mm)	Thickness (mm)
102H	96-102	118	16
106H	100-106	122	16
110H	104-110	127	16
114H	108-114	131	16.2
118H	112-118	136	16.2
122H	116-122	140	16.2
126H	120-126	145	16.2
130H	124-130	149	16.2
134H	128-134	154	16.2
138H	132-138	159	16.7
142H	136-142	163	16.7
146H	140-146	168	16.7
150H	144-150	173	17.2
154H	148-154	177	17.2
158H	152-158	182	17.2
162H	156-162	186	17.8
166H	160-166	191	17.8
170H	164-170	195	17.8

**CHART 19: EasyShoe Versa Grip Glue & EasyShoe Versa Grip Octo (sizes 102-150 only)**

Front Sizes		
Size	Width (mm)	Length (mm)
90F	86-90	98
94F	90-94	102
98F	94-98	106
102F	96-102	110
106F	100-106	114
110F	104-110	119
114F	108-114	123
118F	112-118	128
122F	116-122	132
126F	120-126	136
130F	124-130	140
134F	128-134	145
138F	132-138	150
142F	136-142	154
146F	140-146	158
150F	144-150	163

Hind Sizes		
Size	Width (mm)	Length (mm)
90H	86-90	102
94H	90-94	106
98H	94-98	110
102H	96-102	115
106H	100-106	119
110H	104-110	124
114H	108-114	128
118H	112-118	133
122H	116-122	137
126H	120-126	142
130H	124-130	146
134H	128-134	151
138H	132-138	156
142H	136-142	160
146H	140-146	165
150H	144-150	170

**CHART 20: EasyShoe Versa Grip Wedge**

Front Sizes		
Size	Width (mm)	Length (mm)
<b>114F</b>	108-114	126
<b>118F</b>	112-118	131
<b>122F</b>	116-122	135
<b>126F</b>	120-126	139
<b>130F</b>	124-130	143
<b>134F</b>	128-134	148
<b>138F</b>	132-138	153
<b>142F</b>	136-142	157
<b>146F</b>	140-146	161
<b>150F</b>	144-150	166

Hind Sizes		
Size	Width (mm)	Length (mm)
<b>114H</b>	108-114	131
<b>118H</b>	112-118	136
<b>122H</b>	116-122	140
<b>126H</b>	120-126	145
<b>130H</b>	124-130	149
<b>134H</b>	128-134	154
<b>138H</b>	132-138	159
<b>142H</b>	136-142	163
<b>146H</b>	140-146	168
<b>150H</b>	144-150	173

**CHART 21: EasyShoe Versa Grip Rocker**

Front Sizes		
Size	Width (mm)	Length (mm)
<b>114F</b>	108-114	126
<b>118F</b>	112-118	131
<b>122F</b>	116-122	135
<b>126F</b>	120-126	139
<b>130F</b>	124-130	143
<b>134F</b>	128-134	148
<b>138F</b>	132-138	153
<b>142F</b>	136-142	157
<b>146F</b>	140-146	161
<b>150F</b>	144-150	166

**CHART 22: EasyShoe 3D**

Front Sizes		
Size	Width (mm)	Length (mm)
<b>118F</b>	112-118	128
<b>122F</b>	116-122	132
<b>126F</b>	120-126	136
<b>130F</b>	124-130	140
<b>134F</b>	128-134	145

Hind Sizes		
Size	Width (mm)	Length (mm)
<b>110H</b>	104-110	124
<b>114H</b>	108-114	128
<b>118H</b>	112-118	133
<b>122H</b>	116-122	137
<b>126H</b>	120-126	142